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Advisory Notice

To Whom it may concern,

We, Shenzhen Mindray Scientific Co., Ltd., (Mindray) manufacture of the Mindray BeneFusion eSP syringe pump, with registered business office at 6/F, Bldg2, 1203 Nanhuan Avenue, Yutang Block Guangming District (518106 Shenzhen) PEOPLE'S REPUBLIC OF CHINA., hereby declare that:

Mindray has identified that the lock screen of BeneFusion eSP syringe pump (Configured PCA) may be invalid in some specific scenarios. This effect is limited to software version V01.27 in conjunction with PCA mode.

Therefore,

- If you are using V01.27 software version, safety or performance features other than PCA mode are not affected.
- The device installed with any other software version is not affected.

Note: Special scenes refer to tapping the screen multiple times in rapid succession or operating hard buttons multiple times in rapid succession.

After you have received this notice, please discontinue to use to the BeneFusion eSP syringe pump with PCA mode until the software is updated to 01.28 or above. During this period, the BeneFusion eSP syringe pumps could be used continuously. (The details are in appendix1). Mindray service team will correct the problem as soon as possible.

Contact Information:

We apologize for the inconvenience caused by this situation. If you have any questions, please contact with your local Mindray Customer Service Engineer or designated Technical Support Engineer – Jeffrey.tang

Tel: +86 185 8821 6354

Email: jeffrey.tang@mindray.com

Signature: Emily Zhao

Post-market Surveillance Quality Assurance

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China Council for the Promotion of International Trade
China Chamber of International Commerce

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THIS IS TO CERTIFY THAT: the seal of Shenzhen Mindray Scientific Co., Ltd. on the annexed POWER OF ATTORNEY is genuine.

China Council for the Promotion
of International Commerce

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Authorized
Signature: Lyu Cuilian

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(Date: Jun. 08, 2022)

POWER OF ATTORNEY

Place of issue SHENZHEN

Date of issue May 22, 2022

Company **Shenzhen Mindray Scientific Co., Ltd.** (located at: 6/F, Bldg 2, 1203 Nanhuan Avenue, Yutang Block, Guangming District, 518106 Shenzhen, PEOPLE'S REPUBLIC OF CHINA), hereinafter referred to as "**Manufacturer**", in the person of Legal Representative Ms. Guo Yanmei, acting by virtue of Articles of Association

hereby authorizes:

Company LLC "**Mindray Medical Rus**" (129110, Moscow, intracity territory of a city of federal significance, Meshchansky municipal district, 16 Olimpiskyi prospect, 5 bld., mezzanine floor/apartment. 4/I, 7 flat, 11 A registered on 30, October 2007 under registration number 1077761878908), hereinafter referred to as "**Authorized Manufacturer's Representative**", represented by Wei Xiong, General Director, acting by virtue of the Articles of Association,

to represent interests of the Manufacturer on the questions of circulation of medical devices manufactured by the Company **Shenzhen Mindray Scientific Co., Ltd.** (P.R. China) in the territory of the Russian Federation, and also to make registration, re-registration, certification, recertification, conformity to standards, changes to the registration certificates and other documents, replacement of forms (blank) registration certificates and other documents on medical devices and other products.

to make technical tests, toxicity studies, clinical investigation, expert examination of quality, effectiveness and safety of the medical devices, importation to the territory of Russian Federation, exportation from the territory of Russian Federation, confirmation of compliance with the regulatory, technical and (or) operational documentation of the Manufacturer, and also to perform repair, disposal or destruction of the medical devices.

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6/F, Bldg 2, 1203 Nanhuan Avenue, Yutang Block, Guangming District, 518106 Shenzhen, P. R. China

Website: www.mindray.com

Within the powers stated below, the Manufacturer gives to the Authorized Representative the rights to perform the following activities:

- represent the Manufacturer (its manufacturing facilities, or affiliate companies), in any federal executive authorities of the Russian Federation that perform state registration and control of medical devices circulation, and also in any other Russian organizations engaged in necessary tests, studies, expert assessment of quality, effectiveness and safety of medical devices and other products;
- hold negotiations;
- organize carrying out of necessary studies and tests, including importation of specimens pursuant to the established procedure, within the Russian Federation territory;
- sign applications, requests, agreements and any other necessary documents, including financial ones;
- form and approve technical, operational documents, as appropriate, any other documents, as well as to introduce relevant amendments to the above mentioned documentation;
- provide any technical, operational and other documents and materials, to give any supplementary information and clarifications;
- initiate any necessary information and documents for the purposes of state registration and medical devices conformity certification and other goals;
- pay for any services;
- perform any other needed actions related to medical devices registration or conformity certification and other goals;
- receive and submit any test reports, expert opinions, registration certificates, certificates of conformity, declarations of conformity specimens import permits and other necessary documents;

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6/F, Bldg 2, 1203 Nanhuan Avenue, Yutang Block, Guangming District, 518106 Shenzhen, P. R. China

Website: www.mindray.com

明
瑞
PI
MOTI

mindray

- authorizes to sign and to attest the accuracy, correctness and authenticity of text contents for any translations to Russian of technical and operational documents;
- perform any other needed actions related processes listed above;
- the Registration Certificate will be executed in the name of **Shenzhen Mindray Scientific Co., Ltd.** (P.R. China).
- other documents may be in the name of **Shenzhen Mindray Scientific Co., Ltd.** (P.R. China) or LLC "Mindray Medical Rus (Russia)

This Power of Attorney was issued for the period of **5** (five) years, with the right of substitution.

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