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2 Information on submitter of the report
<b>Status of submitter</b> <input checked="" type="radio"/> Manufacturer <input type="radio"/> Authorised Representative within EEA and Switzerland <input type="radio"/> Others: (identify the role)

3 Manufacturer information	new
<b>Name</b> Aesculap AG	
<b>Contact Name</b> Jacqueline Liebers	
<b>Address</b> Postfach 40	
<b>Postcode</b> 78501	<b>City</b> Tuttlingen
<b>Phone</b> +49 7461 95 31830	<b>Fax</b>
<b>E-mail</b> vigilance_aag.de@aesculap.de	<b>Country</b> DE - Germany

88094  
26.09.2025

**6 Medical device information**

new

<b>Class</b> <input type="radio"/> AIMD Active implants <input type="radio"/> MDD Class III <input type="radio"/> MDD Class IIb <input checked="" type="radio"/> MDD Class IIa <input type="radio"/> MDD Class I <input type="radio"/> IVD Annex II List A <input type="radio"/> IVD Annex II List B <input type="radio"/> IVD Devices for self-testing <input type="radio"/> IVD General	
<b>Nomenclature system (preferable GMDN)</b> GMDN	<b>Nomenclature code</b> 37148
<b>Nomenclature text</b> Laparoscopic access cannula, reusable	
<b>Commercial name/ brand name / make</b> REUSABLE TROCAR SYSTEM 10MM, 12MM	
<b>Model number</b>	<b>Catalogue number</b> EK087P; EK085P; EK083P; EK086P
<b>Serial number(s)</b>	<b>Lot/batch number(s)</b> EK083P - 52978946 EK083P - 52991996 EK083P - 53002275 EK083P - 53004669 EK083P - 53006807 EK083P - 52941286 EK083P - 52956527 EK083P - 52956740 EK083P - 52956952 EK083P - 52957393 EK083P - 52967167 EK085P - 52987668 EK085P - 52993442 EK085P - 53001930 EK085P - 53008562 EK085P - 53013543 EK085P - 52948710 EK085P - 52954233 EK085P - 52957528 EK085P - 52966111 EK085P - 52979586 EK086P - 52968398 EK086P - 52975386 EK086P - 52988022 EK086P - 52993532 EK086P - 53008999 EK086P - 53009000 EK086P - 53009001 EK086P - 53009417 EK087P - 52974276 EK087P - 52952133 EK087P - 52954633 EK087P - 52959357 EK087P - 52964442 EK087P - 52968887 EK087P - 52972214 EK087P - 52977427 EK087P - 52981208 EK087P - 52985519 EK087P - 52988828 EK087P - 52992514 EK087P - 52993617 EK087P - 52997919 EK087P - 53002094 EK087P - 53004347 EK087P - 53006380 EK087P - 53007625 EK087P - 53010869 EK087P - 53013557

## 7 Description of the FSCA

### Background information and reason for the FSCA

Aesculap AG identified that the yellow sealing component of the affected REUSABLE TROCAR SYSTEM 10MM, 12MM exhibits uneven edges with visible silicone fragments. There is the potential risk that silicone fragments detach from the yellow sealing component during use.

Due to the described product problem, there is a potential risk that the silicone fragments could fall in situ when an instrument is inserted through the yellow sealing component. In case the fragments detach intraoperatively, they will likely remain unnoticed due to the restriction of the field of view due to the endoscopic procedure and the small size of the fragments. In case suction devices are applied at the site where the fragments are located, there is the possibility that they may be retrieved.

The short-term hazard posed by the unsighted fragments is considered negligible due to the small weight force of the fragments. Yet, the fragments might lead to long term health consequences resulting from the fragments serving as a foreign body stimulus. They pose a risk for:

- Inflammation or infection
- Encapsulation or adhesion formation

Within the past five years (August 15, 2020 - August 14, 2025), three product complaints related to the RESUABLE TROCAR SYSTEMS 10MM, 12MM were reported to the manufacturer, all involving the same described product issue. The actual occurrence rate of 0.004% exceeds the maximum acceptable limit of 0.001%, as defined in the current product risk analysis. As a result, the associated risk is considered not acceptable.

The clinical impact on the patient depends on several variables such as the severity of the symptoms, the affected organs or tissues, and available treatment options. Typical symptoms of foreign body reactions include swelling and/or pain at the site of the foreign body, fever, fatigue, and swollen lymph nodes.

In a worst-case scenario, albeit remote, a foreign-body reaction can lead to permanent tissue damage. For instance, encapsulation or adhesion formation in immediate vicinity to the ureters can lead to kidney damage in untreated cases of clinically significant (sustained) ureteral obstruction.

Furthermore, patients with a silicone allergy are most at risk as they will display more pronounced foreign body reactions if exposed to the material compared to the general population.

The potential patient harm to patients is therefore classified as "critical".

### Description and justification of the action (corrective / preventive)

Aesculap AG as manufacturer has voluntarily decided to recall the affected products as a precaution due to the risk scenario mentioned in section "Background information and reason for the FSCA".

### Advice on actions to be taken by the distributor and the user

Customers are required to follow the instructions outlined in the Field Safety Notice to identify the affected products. Identified products must be returned to the manufacturer.

### Progress of FSCA , together with reconciliation data (Mandatory for a Final FSCA)

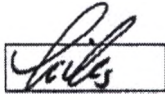
### Time schedule for the implementation of the different actions

The Aesculap AG plans to complete this FSCA within the next 12 months.

**8 Comments**

*Submission of this report does not, in itself, represent a conclusion by the manufacturer and/or authorised representative or the National Competent Authority that the content of this report is complete or accurate, that the medical device(s) listed failed in any manner and/or that the medical device(s) caused or contributed to the alleged death or deterioration in the state of the health of any person.*

Signature

I affirm that the information given above is correct to the best of my knowledge

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